

ASEPT® 2,000 mL Drainage Bag

Maximizing Volume and Expanding Options



- ASEPT® Connector attached to end of extended drain line for easy and fast drainage with the ASEPT® Catheter
- Large 2,000 mL volume drainage bag
- Convenient fluid release valve for bag emptying

ASEPT® 2,000 mL Drainage Bag

The ASEPT® 2,000 mL Drainage Bag and Drainage Bag Kit is the newest addition to the ASEPT® product line and can be used for peritoneal drainage in conjunction with the ASEPT® Peritoneal Drainage System.

This 2,000 mL bag is packaged sterile, ten units to a case and features an extended drain line with the ASEPT® Connector attached to the end of the line for easy catheter connection and drainage, and a pull valve on the bottom of the bag for easy fluid release and evacuation. This bag kit is an economic alternative to a vacuum bottle and can only be used for peritoneal drainage in conjunction with the ASEPT® Peritoneal Drainage Catheter since it relies on gravity rather than vacuum for fluid drainage.

The ASEPT® Drainage Bag comes in two configurations: A Drainage Bag with drain line only and a Drainage Bag Kit. The ASEPT® Drainage Bag Kit is a complete kit that includes the procedure pack with all the necessary dressing change items.

Ordering Information:

ITEM	Ref Number	Case Qty.
ASEPT® 2,000 mL Drainage Bag	622280	10
ASEPT® 2,000 mL Drainage Bag Kit w/procedure pack: One CSR Wrap Containing: Three Alcohol Pads Two Pairs of Gloves Six gauze pads, 4"x4" One Foam Catheter Pad One Self Adhesive Dressing One Blue Emergency Slide Clamp	622281	10

NOTE: When drainage is performed by patients/caregivers or in a setting outside the hospital where standard dressing change items are not readily available, the ASEPT® Drainage Bag Kit must be used.

The ASEPT® 2,000 mL Drainage Bag or Drainage Bag Kit is an accessory to the ASEPT® Peritoneal Drainage System and is to be used for peritoneal drainage with the ASEPT® Peritoneal Drainage Catheter only.

Indications for Use – ASEPT® Peritoneal Drainage System

The ASEPT® Peritoneal Drainage System is indicated for periodic drainage of recurrent and symptomatic malignant ascites. The catheter is intended for long term access of the peritoneal cavity in order to relieve symptoms such as dyspnea.

Refer to The ASEPT® 2,000 mL Instructions for Use for relevant warnings, precautions, complications and contraindications. This device has been designed for single use only.


Rx only

CV2001-8/12

ASEPT® is a registered trademark of pfm medical, inc.
©2012 B. Braun Interventional Systems Inc.

LATEX Latex free
Components and packaging do not contain natural rubber latex

ASEPT® 2,000 mL Drainage Bag

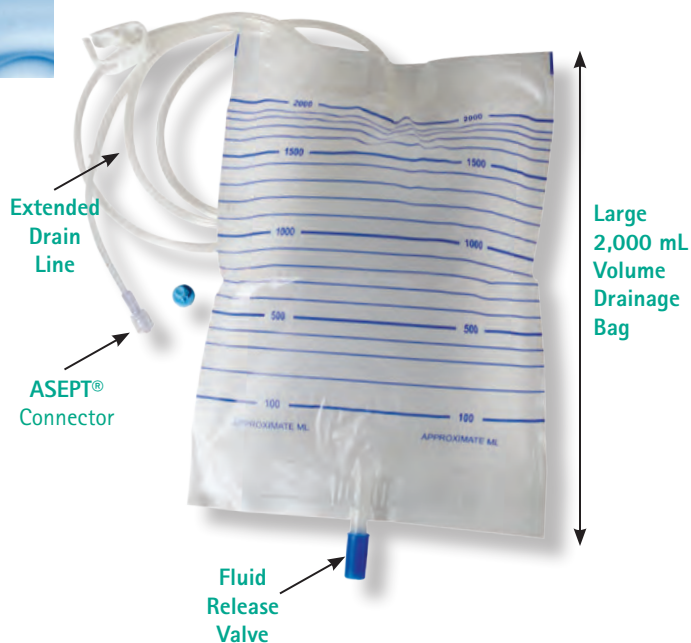

Procedure Pack:



For more information or to place an order, contact your B. Braun Interventional Systems Inc. representative or call 1-877-VENA-CAV (836-2228)



Manufacturer:
pfm medical, inc.
1815 Aston Ave.,
Suite 106
Carlsbad, CA 92008

Manufactured for:
B. Braun Interventional Systems Inc.
824 Twelfth Avenue
Bethlehem, PA 18018 USA
Tel: 1-877-VENA CAV (836-2228) (USA)
Fax: 1-610-849-1334
www.bisusa.org