VenaTech® LP

Brachial Introducer System (Antecubital)

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VenaTech® LP

Brachial Introducer System (Antecubital)

[For US market only: Vena Cava Filter
Brachial Introducer System (Antecubital)]⁽¹⁾

REF. 04439985

Instructions for use for implantation
via the **brachial** approach

Use this document only if you have chosen to implant
the VenaTech® LP filter from the Brachial approach (Antecubital).
Do not use these instructions for use for any other approach
or for the implantation of any other vena cava filter.

B | BRAUN

INTRODUCTION

The VenaTech® LP Brachial Introducer System (Antecubital) must be used with a VenaTech® LP vena cava filter only.

The VenaTech® LP Brachial Introducer System (Antecubital) includes a **96 cm long (37.8 in.)** sheath, designed to be used during the implantation of a VenaTech® LP vena cava filter via the Brachial approach (Antecubital).

Please refer to the VenaTech® LP instructions for use booklet (JUGULAR, or SUBCLAVIAN for OUS only, approaches) for information about the use of a VenaTech® LP vena cava filter.

SYSTEM CONTENTS

- 7F I.D. Introducer sheath with hemostasis valve and side connection
- 7F Dilator
- 7F Long Pusher (white)
- 7F Short Pusher (blue)
- PTFE coated "J" guide, length = 180cm, diameter = 0.89mm (0.035 in)

For an implantation of a VenaTech® LP vena cava filter from the Brachial approach (Antecubital), it is mandatory to use the above-mentioned accessories, instead of the accessories available into the VenaTech® LP vena cava filter kit. The only items that must be used from the VenaTech® LP vena cava filter kit are the following :

- Filter in its cartridge
- Cartridge cradle
- Instructions for use (JUGULAR, or SUBCLAVIAN for OUS only, approaches)

INDICATIONS

The VenaTech® LP Brachial Introducer system (Antecubital) is intended for use with the VenaTech® LP Vena Cava Filter.

IMPLANTATION PROCEDURE

Please refer to the VenaTech® LP instructions for use booklet (JUGULAR, or SUBCLAVIAN for OUS only, approaches) for important warnings / indications / contraindications / implantation procedure of the VenaTech® LP Brachial Introducer System (Antecubital). [For US market only: Vena Cava Filter Brachial Introducer System (Antecubital)]⁽¹⁾

Note for the implantation from the Brachial approach :

- **Implantation approach:**
This Introducer System is designed for insertion via the **Brachial approach (Antecubital)**. If needed, this Introducer System can be used to insert the filter via the right internal jugular vein or the left or right subclavian veins. In this case, due to the length of the introducer, it is important to rigorously follow aseptic technique.
Other approaches (particularly the left internal jugular vein, the right or left external jugular veins) should not be attempted.
- **Preparation of the patient:**
Stretch the arm of the patient 90 degrees from the long axis of the spinal column. This will allow you to place the introducer in the axis of the subclavian vein.

PRECAUTIONS

During the implantation via the Brachial approach (Antecubital) the filter cartridge must be inserted in the JUGULAR position into the cartridge cradle.

Introducer system for use with VenaTech® LP vena cava filter for permanent interruption of the Inferior Vena Cava. STERILE and non-pyrogenic, sterilized using Ethylene Oxide. Contents of unopened, undamaged package are sterile and non-pyrogenic. **For single use only.** The device and its accessories are not re usable nor designed to be re-used. Any re-use would definitely compromise the performance and safety of the device. Destroy system components after filter implantation. Components may not be cleaned, resterilized or reused. Store at room temperature. Avoid freezing and excessive heat. Do not remove from unit carton until ready for use.

Caution : Federal (U.S.A.) Law restricts this device to sale by or on the order of a physician.

Opening the sterile packaging :

Confirm the date of sterile expiration on the labelling. Never use an introducer system which has exceeded its sterile expiration date. To maintain sterility, all normal operating room sterile procedures should be followed.

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B | BRAUN

For US market only:⁽²⁾

Manufactured for:⁽³⁾
B. Braun Interventional Systems Inc.
824 Twelfth Avenue
Bethlehem, PA 18018
www.bisusa.org

Customer Service, ordering:⁽⁴⁾
Tel: (877) VENA-CAV (836-2228)
Fax: (610) 849-1334

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